

HANDLER

RED WING INTERNATIONAL

MODEL #750
SANI-VAC



 Made in the U.S.A. Since 1920

INSTRUCTION MANUAL

DO NOT DISCARD
WARNING!

FOR YOUR SAFETY PLEASE READ INSTRUCTIONS
BEFORE OPERATING TOOL. & WEAR EYE PROTECTION



INSPECTION

Open the container in which the Sani-Vac was shipped and inspect it. Should damage have occurred in transit, contact the freight carrier who delivered the Sani-Vac immediately.

Failure to contact the freight carrier who delivered your unit within 15 days will result in possible loss of your rights to claim receipt of a damaged product. Your Sani-Vac, was tested and properly packaged at our manufacturing facility. Should you experience difficulty with the Sani-Vac, contact your distributor immediately.

Description of 750 Parts & Accessories

1. On/Off Button
2. Cord Rewind
3. Variable Suction Control
4. Stabilizer Mount
5. Hose inlet
6. Full bag indicator
7. Stainless Flex Hose
8. Inlet Flange
9. Resin Inlet Adapter

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING THIS APPLIANCE

WARNING: The manufacturer cannot accept responsibility for damage caused when the appliance is not used according to the instructions, or for uses other than those for which it was intended.

To reduce the risk of fire, electric shock or injury:

- Do not leave appliance when plugged in. Unplug from outlet when not in use and before servicing.
- **WARNING!** Electric shock could occur if used outdoors or on wet surfaces.
- Do not allow to be used as a toy. Close attention is necessary when used by or near children.
- Use only as described in this manual. Use only manufacturer's recommended attachments.
- Do not use with damaged cord or plug. If appliance is not working as it should, has been dropped, damaged, left outdoors, or dropped into water, return it to your retailer for service.
- Do not pull or carry by cord, use cord as a handle, close a door on the cord, or pull cord around sharp edges or corners. Do not run appliance over cord. Keep cord away from heated surfaces.
- Do not unplug by pulling on cord. To unplug, grasp the plug, not the cord.
- Do not handle plug or appliance with wet hands.
- Do not put any object into openings. Do not use with any opening blocked; keep free of dust, lint, hair and anything that may reduce air flow.
- Keep hair, loose clothing, fingers and all parts of the body away from openings and moving parts.
- Do not pick up anything that is burning or smoking, such as cigarettes, matches or hot ashes.
- Do not use without vacuum bag and/or filters in place.
- Turn off all controls before unplugging.
- Use extra care when cleaning on stairs.
- Do not use to pick up flammable or combustible liquids such as gasoline or use in areas where they may be present.
- Do not attempt to service the unit while appliance is plugged in.

SAVE THESE INSTRUCTIONS

Handlermfg.com | 750 SaniVac Instruction Manual | 2012

Replacing the filters

It is recommended the vacuum bag is changed when the full bag indicator changes color or the suction power of the canister becomes weak.

1. Turn off the canister
2. Remove the hose and unplug it from the canister body.
3. Open the dust cover.
4. Remove the entire bag holder. **DO NOT THROW IT AWAY** (See Fig. 1)
5. Discard the full bag.
6. Insert a new bag by aligning the notch on the bag collar with the base of the bag holder. (See Fig. 2)
7. Lock the bag into place under the bag holder tab.
8. Insert the bag holder by sliding the 2 prongs back into the canister body. (See Fig. 1).
9. Replacement Filter bag 12/Set Order 750F-FB
10. Close the dust cover.

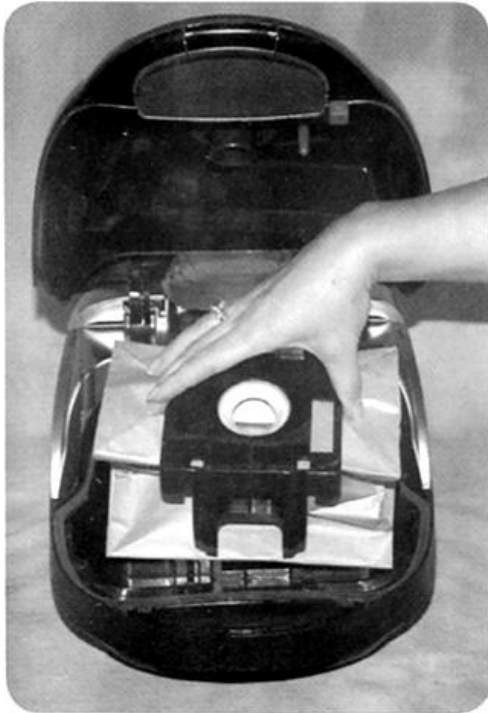


Fig. 1

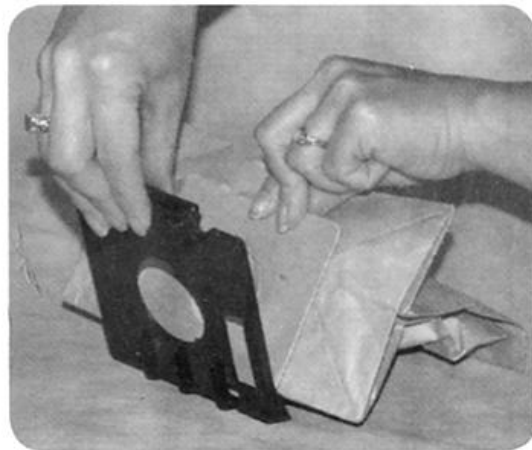


Fig. 2

1. **Replacing the secondary filter**

Turn off the canister and remove the hose. Open the dust cover. Remove the filter from behind the bag.

(See Fig. 3)

Either shake the dust out of the filter or wash it by hand. (Make sure the filter is completely dry before reinstalling.)

Reinsert the filter and close the dust cover.

Replacement Filter: Order 750PF



Fig. 3

2. **Replacing the HEPA filter**

Turn off the canister and power nozzle. Remove the hose and unplug it from the canister body. Open the dust cover.

Remove the filter from the top of the canister by pulling the tab on the filter toward you. (See Fig. 4)

Discard the old filter.

Reinsert a new filter by sliding it back into the holder. Close the dust cover.



Replacement Filter:

Order 750F-FB (Set of 12)



Fig. 4

ONLY WHEN NECESSARY

To tighten stainless steel Sani-Vac hose, grasp the hose (not by the end fittings). Twist firmly with right hand in this direction  , counter clockwise while twisting left in the opposite direction clockwise  . Any part, or all of the hose may be adjusted. Insert cuff firmly into Sani-Vac opening.

DO NOT OVERWIND!

Troubleshooting Guide

Problem	Check Point	Solution
Suction power is weak.	Vacuum bag	If bag is full, replace it.
	Wand or hose	If clogged or blocked remove the object
Vacuum does not work	Is the hose adapter inserted into the vacuum completely?	Insert it completely
	When the vacuum bag is full or if the wand or hose is blocked, the motor thermal protection is activated.	Unplug the vacuum and check the bag, wand and hose and remove any blockages. The vacuum can be turned on again after approximately 20 minutes
Power cord does not rewind	Power cord	Pull the power cord out a little and press the cord rewind button again
Power cord won't pull out of the vacuum	Power cord	Do not pull the cord by force. Rewind it a little by pressing the cord rewind button and try pulling it out again.

Polarization Instruction

QuickTime™ and a decompressor are needed to see this picture.

Power Cord, 2-Wire Polarized Cord

Household model FB-SSCAN

To reduce the risk of electric shock this product is equipped with a polarized alternating current line plug (a plug having one blade wider than the other). This plug will fit into the power outlet only one way. This is a safety feature. If you are unable to insert the plug fully into the outlet, try reversing the plug. If the plug should still fail to fit, contact a qualified electrician to replace the obsolete outlet. Do not defeat the safety purpose of the polarized plug.

IMPORTANT NOTICE

WHAT SHOULD YOU DO TO RETURN GOODS

1. Our terms of sale are F.O.B. our plant. Handler makes every effort to insure the proper delivery of goods, but we cannot guarantee the carrier's performance. Each product is tested and inspected for proper performance prior to leaving our facility. The responsibility for damage in transit is the carrier's, whether it is visible damage or concealed damage.
2. Inspect this shipment **IMMEDIATELY**. Insist that visible damage or possible damage be written on the Delivery Receipt by the receiver.
3. Inspect the contents of your shipment within 24 hours of receipt. Using your copy of our packing list, check for piece count and accuracy immediately to insure proper billing.
4. If damage is discovered, save all crates, cartons and packaging material until an inspection has been made by an agent of the carrier.
5. In case of damage, notify the delivering carrier **IMMEDIATELY** requesting that an inspection be made. Retain a copy of the inspection request for claim purposes. Failure to notify the freight carrier immediately will result in the loss of your rights to claim damage.
6. We have taken every precaution to insure safe arrival of your products. We cannot be responsible for negligence by another party.
7. The receiver of the goods must make claims for incorrectly shipped goods to the shipper within 15 days of receipt of delivery.

RETURNING GOODS

1. **DO NOT** return merchandise without our authorization. You must call first for a return authorization number.
2. We will not accept returned merchandise from the carrier unless our authorization **NUMBER** has been issued, and is clearly marked on the outside of the carton.
3. We will not accept goods (i.e. dust collectors) that are not cleaned and free of dust and debris. The merchandise will be shipped back.

HANDLER MFG.
612 NORTH AVENUE EAST, WESTFIELD, NEW JERSEY 07090

Handlermfg.com | 750 SaniVac Instruction Manual | 2012

WARRANTY

This HANDLER product is warranted to be free from defects in material and workmanship, when used under the conditions recommended by the manufacturer, and/or the purpose for which the product was intended, for a period of 12 months from the date of shipment from factory to job site of original owner. Products purchased from distributor warehouse stock are warranted for a period of 18 months from date of shipment from the manufacturer's warehouse.

In all cases, the full name and address of the distributor must be supplied, along with the name of the customer, product name and number, and serial number. A copy of the original invoice, showing date of purchase must accompany the warranty repair request. This warranty will apply to equipment installed, operated and maintained in accordance with HANDLER procedures and recommendations.

During the life of this warranty, HANDLER will repair or replace (At HANDLER's option) free of charge. F.O.B. its plant, any defective part or assembly, if such defect occurred in normal service and was not due to apparent misuse, abuse, or accident. Any warranty service performed in the field must be authorized by HANDLER MFG. Unauthorized service voids the warranty and any resulting charge will not be paid by HANDLER MFG.

HANDLER MFG. makes no other warranties or guarantees, expressed or implied. The merchantability of the components is expressly excluded. The manufacturer assumes no liability for indirect or consequential damages.

.....
Detach or copy and mail back to the address below.
.....

WARRANTY FORM

Customer Name:		
Address:		
City:	State:	Zip Code:
Description of Product		
Model Number:		
Distributor:		
Distributor's Address:		

HANDLER MFG.
612 NORTH AVENUE EAST, PO BOX 520
WESTFIELD, NEW JERSEY 07090-0520